



## Effective cardiac massage

Resuscitation for the professional sector

**animax  
mono**

[www.animax-cpr.de](http://www.animax-cpr.de)

Continuously performing cardiac massage is often a great challenge. Long-lasting resuscitation after thrombolysis therapy or resuscitation in a driving ambulance is often physically strenuous for professional paramedics.

Animax mono was developed particularly for the professional sector to facilitate a long-lasting cardiac massage. A lever boosts the power which results in an easy operation for the user.

Animax mono is always ready for operation because it is independent of any energy source.

Animax mono adjusts to a pressure depth between 40 mm and 50 mm depending on the patient's chest size.

This automatic adjustment provides great protection to avoid the breaking of ribs.

The symmetrically adjustable portal frame makes it possible to quickly put the device on the patient. It is always centred over the patient's chest.

Animax mono is fixed by the scoops that are slid sideways under the patient's back.

The cardiac massage is performed exactly from above. The pressure stamp sits solidly on the chest because it rotates with the application of power, while, on the other hand, during the decompression phase the chest is fully relieved.



During defibrillation the animax mono does not need to be removed from the patient.



Ideen bewegen mehr

# Innovative technology for effective cardiac massage — the animax mono relieves the paramedic tremendously

Our goal at AAT is to provide resuscitation aids based on new ideas and experience.

Our entire know-how and skill based on great experience in rehabilitation technology we bring into emergency medicine. Accordingly the animax mono is designed to solve complex problems with mobile and user-friendly technology.

- cardiac massage with little physical effort
- independent of an energy source
- the pressure depth is self-adjusting, depending on the patient's chest size
- quick and easy localization of the pressure point due to a pressure stamp
- complete relief of the chest during decompression
- does not have to be removed during defibrillation
- quickly put on the patient due to the symmetrically adjustable portal frame

animax-mono\_gb\_04\_2009



## Technical and medical data

Target group	adults
Operation	one person
Weight of the animax	8.0 kg
Weight of the animax mono's bag	1.8 kg
Assembly time	approx. 5 seconds
Depth of pressure	40 - 50 mm



Ideen bewegen mehr

AAT Alber Antriebstechnik GmbH  
Postfach 10 05 60 · D-72426 Albstadt

Tel.: +49.74 31.12 95-0 · Fax: 12 95-35  
info@aat-online.de · www.aat-online.de